

1	In the main window display, press the multifunction button next to the Return Arrow key as many times as needed to bring you to the final menu page.	100:17:43     Image: Second seco
2	Press the Auto CasePush quick access key	
3	Confirm by pressing "Continue" in the pop-up window. This will close the current case and uploads the last 5 cases, unless already done. If this procedure is done after each call, there will only be the most recent case to upload	2021 61 23 171 18.10 Add The Data Control of Control o
4	You will see the "Disclosure Log Transfer" window and the progress of the CasePush.	NOT 41-00 17 16.00 And 11   Image: Construct Construct Image: Construct Construct Image: Construct Construct   Image: Construct Construct 2 of 6   Current Construct Image: Construct   Image: Construct Construct Image: Construct
5	Once completed the Green transmission LED will turn off and the device will power off	